



PACEY CUFF

ACTIVEGUARD
REUSABLE INCONTINENCE PAD

INSTRUCTIONS FOR USE

For light to medium incontinence



REF

PCG-01



Comfortably Stop Leaks, Protect Circulation.

Made in Canada

INDICATIONS FOR USE

The Pacey Cuff ActiveGuard is a reusable absorbent pad with a waterproof breathable layer and it is intended for use with the Pacey Cuff Ultra urethral control device.




INSTRUCTIONS FOR USE

1. Position the ActiveGuard in form-fitting underwear to absorb any light urinary incontinence.
2. If you experience intermittent leakage while using the Pacey Cuff: Use the ActiveGuard without any extra pad or adsorbent element.
3. If you experience slow and/or steady leakage while using the Pacey Cuff: try tightening it a bit by adjusting the bottom a bit more towards the 4 and then double check its position under the urethra.

CLEANING

The ActiveGuard can be washed and dried with your regular clothing. Do not use bleach or stain removing compounds. Alternatively, hand-wash with hot water and a mild non-hydrating soap.

For quicker drying, compress and squeeze excess water from the Pacey Cuff ActiveGuard with a towel.

Symbol	Definition
	Manufacturer
	Consult instructions for use
	Product code

PACEY MEDTECH LTD. 203-1099 W 8th Ave. Vancouver, BC V6H 1C3 Canada

paceycuff.com | Tel: 1-877-377-2239 | info@paceycuff.com

Pacey Cuff is a trademark of Pacey MedTech Ltd. ©2024

