


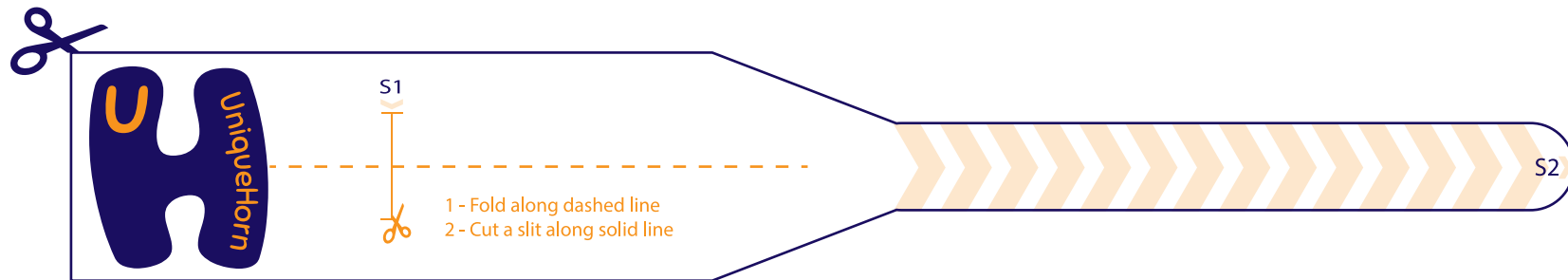

Measuring Band



UniqueHorn Instructions:

- Print A4 Page (UniqueHorn Measuring Band)
- Cut out measuring band
- Fold along the central dashed line
- Cut small slit where marked S1
- Feed the end S2 through the slit S1, to create a measuring band
- Place around your penis when flaccid and tighten band
- Mark on the band using a black fine texter, to create an accurate circumference measurement
- Remove and measure distance between S1 and marker, with a millimetre ruler
- Take that millimetre figure and enter it into your personal details form online, to place an order for your new UniqueHorn.